

DONNING AND DOFFING OVER-THE-HEAD FULL COVERAGE GOWNS

Donning Gown

1




Place your arms in the sleeves with thumbs in thumb loops.

2



Pull the gown over your head.

3

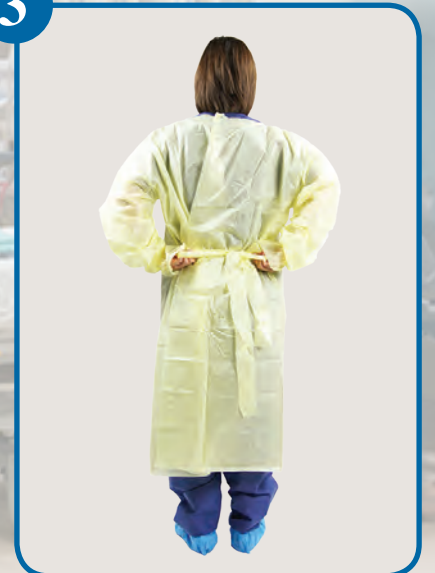

Tie in back at the waist.

DoFFing Gown

1



Pull
Straight
Out and
Down

Grasp the waist at belt area with both hands, and pull straight out and down to break gown from back.

2



Gown can then be pulled away from the body.

3



Roll into a ball to contain any fluids or contamination, then properly dispose the gown.