

Cold Packs

Precept's Patented Ice Bags are available in three sizes, the Premium Cold Packs in large and small sizes, and universal sized Cold Pack. Precept Patented features include a wide mouth with opening aids that allow for extremely easy filling, a clip sleeve provides a secure holding area for the clip, and a bladder that is attached to the outer covering preventing it from slipping down in the bag which helps provide even cold therapy. Once the clip is closed it is secured and surrounded by the outer layers of the ice bag which provides a more comfortable application. Sturdy and strong ties allow for secure attachment. The three-layer design offers a soft outer layer for comfort, an absorbent middle layer, and an impervious inner plastic bladder.



Refillable single patient use only.

| ITEM # | DESCRIPTION | SIZE | COLOR | PACKAGING |
|--------|-------------------|---------------------------|------------|----------------------------|
| 16100 | Premium Cold Pack | Large (8" x 13") | White ○ | 25/Box, 4/Boxes, 100/Cs |
| 16101 | Premium Cold Pack | Small (6" x 13") | White ○ | 25/Box, 4/Boxes, 100/Cs |
| 16105 | Cold Pack | Universal (7.5" x 13") | White ○ | 25/Box, 4/Boxes, 100/Cs |

U.S. Patent No. 7,972,368

INSTRUCTIONS FOR USE

1. Fold down outer covering to expose clip and plastic bladder. Push clip in to open, then apply pressure to the sides of the bladder mouth, so that it forms an "O".
2. Fill with crushed ice to desired level.
3. Purge Air
4. Fold mouth of bladder over clip and fasten clip securely.
5. Tuck the clip inside the outer covering and secure the Velcro closure.
6. Apply to patient.

Precept products do NOT
contain natural rubber latex

CP0021 Rev 3 1303

For more information or to order,
contact Precept at **800.438.5827**
or visit www.preceptmed.com