

PROPER DONNING OF N95 SURGICAL RESPIRATOR

In order to obtain the maximum level of protection from your Precept N95 Respirator, a fit test should be conducted prior to the first use. Fit testing should also be conducted on an annual basis and any time there is a change that might affect fit quality (example: facial plastic surgery, excessive weight gain or loss, etc.)



1
Separate the two headbands.



2
Hold the headbands while you cup the respirator to your face. Pull the headbands over your head.




3
Position the lower headband at the base of your neck; the higher headband should go around the crown of your head.



4
Press firmly on both sides of the nosepiece to create a secure, comfortable fit.



5
If necessary, make minor adjustments until the respirator fit feels secure.

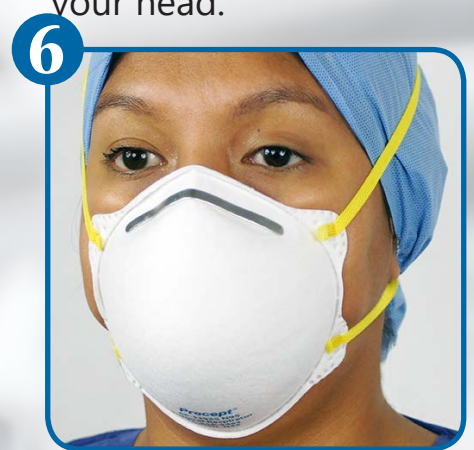

6
Properly donned N95 surgical respirator.