

51602

Isolation Gown

Over-the-Head, thumb loop gown with open back.

- Polycoated material delivers protection.
- Low linting material.
- Over-the-Head donning improves compliance.
- Thumb loops ensure proper glove donning.



Specifications

Product	Over-the-Head, Open Back Isolation Gown
Size	X-Large
Dimensions	Length 48", Width 42"
Color	White
Wrist	Thumb Loop
Material	Polycoated
Design	OTH, Open Back
Seams	Thermal Impulse Sealed
Packaging	10/Bag, 50/Case, Dispenser Box

This product is NOT made with natural rubber latex



Precept makes no representation or warranty that its products will provide absolute protection from contact or exposure to contaminants or blood, or eliminate the risk of contracting diseases of any nature. OSHA requires the employer to evaluate the task and type of exposure anticipated and select the appropriate personal protective equipment (PPE) that reduces the employee's exposure to potential hazards.

DONNING AND DOFFING OVER-THE-HEAD OPEN BACK GOWNS

Donning Gown

1



Place your arms in the sleeves with thumbs in thumb loops.

2



Pull the gown over your head.

3



Tie in back at the waist.

DoFFing Gown

1



Pull the gown forward with both hands breaking away from the back, neck, and waist.

2



Gown can then be pulled away from the body.

3



Roll into a ball to contain any fluids or contamination, then properly dispose the gown.