



50095 Isolation Gown – over-the-head thumb loop gown with open back

Intended Use

Intended to be used in environments where fluid exposure to the back is not a risk.

Technical Specifications

Part No.	50095
Description	Over-the-Head, Open Back Isolation Gown
Color	Yellow
Size	Universal
Dimensions	Length 56", Width 41"
Wrist	Thumb Loop
Material	Multi-Layer SMS
Design	Thumb Loop, Open Back
Seams	Sonic-Welded
Qty	10/Bag, 100/Case

Key Features

- Medium weight multi-layer SMS material^① passes AAMI Level 2 testing and provides comfort
- Over-the-head donning improves compliance
- Thumb loops ensure proper glove donning
- Universal, one size fits most
- Made in North America
- Not made with natural rubber latex

Isolation Gown - over-the-head thumb loop gown with open back

Donning Gown



1 Place your arms in the sleeves with thumbs in thumb loops.



2 Pull the gown over your head.




3 Tie in back at the waist.


Doffing Gown



1 Pull the gown forward with both hands breaking away from the back, neck, and waist.

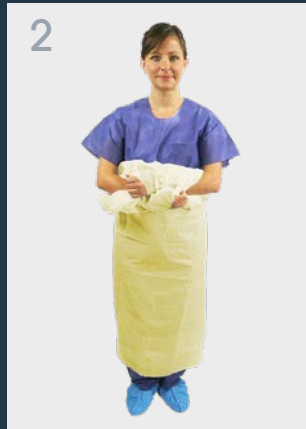

2 Gown can then be pulled away from the body.

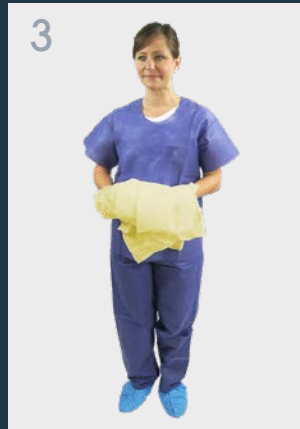

3 Roll into a ball to contain any fluids or contamination, then properly dispose the gown.

ⒶAATCC 42 - Water Resistance: Spray Impact Penetration Test / AATCC 127 - Water Resistance: Hydrostatic Pressure Test For more information on these test methods, please visit aatcc.org and aami.org Precept makes no representation or warranty that its products will provide absolute protection from contact or exposure to contaminants or blood, or eliminate the risk of contracting diseases of any nature. OSHA requires the employer to evaluate the task and type of exposure anticipated and select the appropriate personal protective equipment (PPE) that reduces the employee's exposure to potential hazards. Aspen Surgical reserves the right to make changes without notice in design, specifications and models.

©2020 Aspen Surgical Products, Inc. ALL RIGHTS RESERVED.
PA0336 rev 1 2005 21-SEPT-2020 ENG - US

Precept Medical Products, Inc. an
Aspen Surgical Products, Inc. Company
phone +1 800.438.5827
preceptmed.com

