



51193FC Isolation Gown – over-the-head thumb loop gown with full coverage AAMI level 2

Intended Use

Intended to be used in environments where low to moderate fluid exposure is a risk.

Technical Specifications

Part No.	51193FC
Description	Over-the-Head, Full Coverage Isolation Gown
Color	Yellow
Size	Universal
Dimensions	Length 48", Width 61"
Wrist	Thumb Loop with Elastic Wrists
Material	Multi-Layer SMS
Design	Thumb Loop, Full Coverage
Seams	Sonic-Welded
Qty	10/Bag, 100/Case

Key Features

- Multi-layer SMS material[®] delivers AAMI Level 2 protection and comfort
- Thumb loops with elastic wrist ensure proper glove donning
- Full coverage with overlapping back panels
- Universal, one size fits most
- Made in North America
- Not made with natural rubber latex



Barrier Protection AAMI PB70:2012

Test Method	Test Result	Anticipated Risk of Exposure
AATCC 42 Spray Impact	≤ 1.0g	Low
AATCC 127 Hydrostatic Pressure	≥ 20cm	Low

Isolation Gown – over-the-head thumb loop gown with full coverage AAMI level 2

Donning Gown



1 Place your arms in the sleeves with thumbs in thumb loops.



2 Pull the gown over your head.




3 Tie in back at the waist.


Doffing Gown



1 Pull the gown forward with both hands breaking away from the back, neck, and waist.

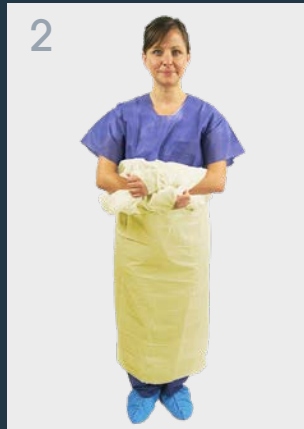

2 Gown can then be pulled away from the body.

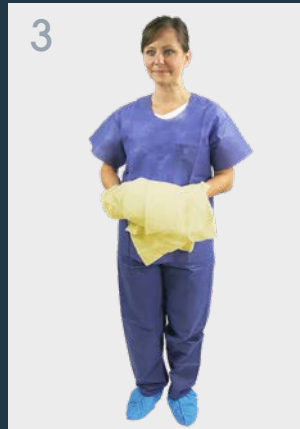

3 Roll into a ball to contain any fluids or contamination, then properly dispose the gown.

①AATCC 42 – Water Resistance: Spray Impact Penetration Test / AATCC 127 – Water Resistance: Hydrostatic Pressure Test For more information on these test methods, please visit aatcc.org and aami.org Precept makes no representation or warranty that its products will provide absolute protection from contact or exposure to contaminants or blood, or eliminate the risk of contracting diseases of any nature. OSHA requires the employer to evaluate the task and type of exposure anticipated and select the appropriate personal protective equipment (PPE) that reduces the employee's exposure to potential hazards. Aspen Surgical reserves the right to make changes without notice in design, specifications and models.

©2020 Aspen Surgical Products, Inc. ALL RIGHTS RESERVED.
PA0096 rev 7 1511 02-OCT-2020 ENG – US

Precept Medical Products, Inc. an
Aspen Surgical Products, Inc. Company
phone +1 800.438.5827
preceptmed.com

